


***Before priming filter ensure RED and WHITE clamps are closed.**

Priming Filter:

- (a) Attached blood bag to filter
- (b) Open red clamp
- (c) Open green air vent.

The blood will then fill the back of the filter first, then will fill front of filter. Once full close green air vent.

- (d) Open white clamp prime remaining section of filter line.

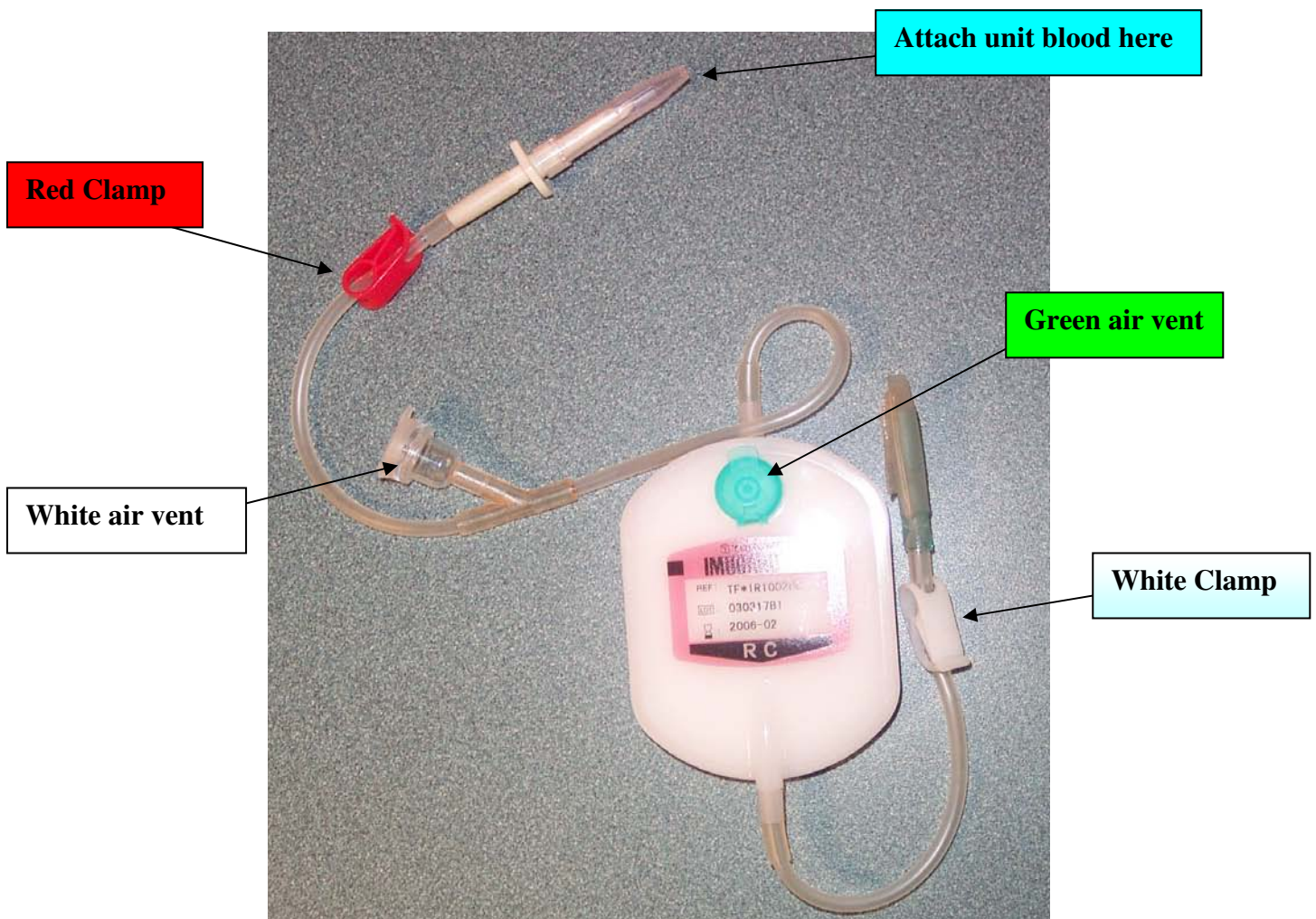

TO EMPTY FILTER ONCE BLOOD HAS FINISHED:

- (a) Close Red Clamp
- (b) Open White Air Vent and blood will empty from back of filter
- (c) Then open Green Air Vent and blood will empty from front of filter and proceed down past the white clamp.

You then can remove empty unit and filter and attach flush or if on pump flush with other line

DO NOT AT ANY TIME FLUSH SPECIAL FILTER WITH NORMAL SALINE. ONLY RED CELLS GO THROUGH THIS FILTER.

