


R3 Lifeblood shipping configuration for Ambulance Victoria emergency use RBC

2 units red blood cells (RBC). Validated transport time: 8 hours 25 minutes

Total chilled ballast required = 14. If 4x RBC required, pack into 2x Lifeblood shippers.

Ballast MUST be chilled at 2-6°C for ≥24 hours prior to use.

Lifeblood shipping configurations available at <https://www.lifeblood.com.au/health-professionals/products/storage-and-handling/transport-of-blood>

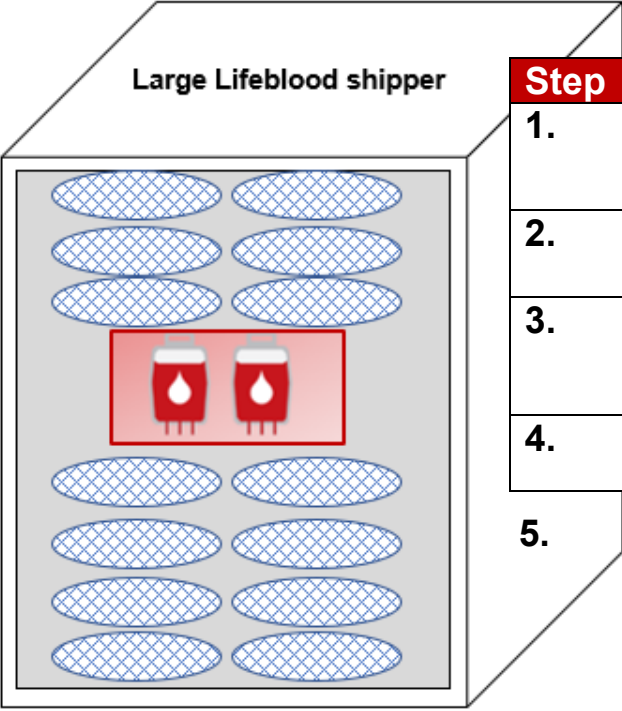

Step	Legend	Name	Quantity	Note
1.		Foil pouch	1	Used to line box and contains all other items
2.		Chilled ballast	8	Inside foil pouch, 4 layers of 2
3.		Plastic liner bag and 2x RBC	1	Used to contain RBC (and temperature logger if used)
4.		Chilled ballast	6	Inside foil pouch, 3 layers of 2
5.	Close foil pouch, seal shipper (including compatibility report if supplied) and attach completed label to top of shipper			